BIO-C[®] TEMP

CHOOSE YOUR LANGUAGE

ENGLISH.....	3	HRVATSKI	41
ESPAÑOL.....	6	SLOVENŠČINA	44
PORTUGUÊS	9	SLOVENSKY	47
FRANÇAIS	12	POLSKI	50
ITALIANO	16	EESTI KEEL.....	53
ROMÂNĂ	19	LIETUVIŲ	56
NEDERLANDS.....	22	ČESKY	59
DEUTSCH	25	MAGYAR	62
SVENSKA.....	29	TÜRKÇE.....	65
DANSK	32	БЪЛГАРСКИ ЕЗИК.....	68
NORSK.....	35	СРПСКИ.....	72
SUOMI	38	ΕΛΛΗΝΙΚΗ	75

ENGLISH DEFINITION

BIO-C® TEMP is a ready-to-use bioceramic paste used as an intracanal dressing prior to root canal filling, and it can also be used in cases of perforation and internal and external resorptions.

INTENDED USE

BIO-C® TEMP belongs to the family of endodontic cements, mainly composed of Mineral Trioxide Aggregate (MTA), intended for use in patients requiring endodontic procedures, serving as an intracanal dressing for endodontic treatment.

INDICATIONS

- Intracanal dressing for endodontic treatment with pulp necrosis, refractory lesions, retreatments, persistent fistulas, and excessive exudate.
- Intracanal dressing in cases of perforations, external and internal resorptions, prior to the use of MTA ANGELUS, MTA REPAIR HP, or BIO-C® REPAIR.
- Incomplete root development: Apexification – for root apex formation in cases of pulp necrosis.

INTENDED POPULATION

Patients in need of endodontic treatment.

INTENDED USER

Dentists trained in endodontic procedures.

INTENDED ENVIRONMENT

Dental office.

CONTRAINDICATION

Do not use in patients with reported sensitivity to the components of the formula.

COMPOSITION

Calcium silicates, calcium aluminate, calcium oxide, resin base, calcium tungstate, dispersing agent, and titanium dioxide.

TECHNICAL INFORMATION

- High alkalinity.
- Adequate radiopacity.

Important: As an intracanal dressing, BIO-C® TEMP is not intended to set within the root canal.

WARNINGS

- The product is moisture-sensitive, so close the packaging properly with adequate pressure to prevent

product drying.

- Do not reuse the applicator tips.
- Clean and disinfect the syringe for subsequent uses to prevent cross-contamination.
- Disinfect the applicator tip prior to use. After use, discard the applicator tip.
- Avoid excessive product extrusion into the periradicular region.

Note: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority health in which the user and/or patient is established.

PRECAUTIONS

- Wear protective goggles, mask, and gloves when handling the product. In case of contact with eyes or skin, rinse with water.
- Do not use the product after the expiration date.
- Do not use the product if the packaging is damaged.
- Product not tested on pregnant women, lactating mothers, and children.
- The product should remain temporarily in the root canal of the patient (not exceeding 30 days).
- Keep out of reach of children.

TECHNIQUE OF USE

A. USE OF APPLICATOR TIPS

1. It is recommended to use a disposable plastic cover on the syringe to prevent cross-contamination.
2. Confirm material flow from the syringe before direct application to the canal.
3. Position the syringe and tip assembly properly in the canal and press the plunger lightly to prevent excessive product extrusion.
4. Before closing the syringe, retract the plunger to stop the flow.
5. Wipe off excess material from the syringe tip and close with adequate pressure after use to prevent moisture ingress, which can cause product drying.
6. Clean and disinfect the syringe for subsequent uses to prevent cross-contamination. In case of contamination with saliva or blood, discard it.

B. INTRACANAL APPLICATION

1. Anesthetize, place rubber dam isolation, and then perform biomechanical preparation of the canal. In cases of internal resorption, use curettes to remove tissue.
2. Dry the canal with absorbent paper points.
3. Remove the sealing cap, dispense material from the syringe tip, which may be slightly hardened.
4. Connect the applicator tip to the syringe.

5. Insert the syringe tip near the root apex.
6. Apply BIO-C® TEMP by gradually retracting the syringe to ensure complete canal filling.
7. Remove excess paste from the pulp chamber.
8. Perform coronal sealing and temporary restoration or temporary sealing. The procedure should be performed according to the instructions for use of the materials used.
9. Take a radiograph to verify proper canal filling.

Important: BIO-C® TEMP should not remain in the root canal of the patient for more than 30 days.

C. MATERIAL REMOVAL:

1. Anesthetize and install rubber dam isolation.
2. Final removal of BIO-C® TEMP before canal filling should be done using Sodium Hypochlorite associated with the use of a K-file with filing motion against the canal walls.
3. Then, irrigate with 17% EDTA solution, and if possible, activate EDTA with an ultrasonic tip alternating with Sodium Hypochlorite in 3 cycles of 10 seconds duration for each irrigating solution.
4. In cases of intracanal dressing for endodontic treatment of teeth with pulp necrosis and retreatments, fill the canal with BIO-C® SEALER, MTA-FILLAPEX, or another preferred obturation material. The endodontic obturation procedure should be performed according to the instructions for use of the material.
5. In cases of intracanal dressing in teeth with perforations, external and internal resorptions, apply MTA ANGELUS, MTA REPAIR HP, BIO-C® REPAIR, or another preferred repair material. The procedure should be performed according to the instructions for use of the product, prior to obturation or dressing change.
6. Seal the coronal opening.
7. Take radiographs and perform follow-up.

STORAGE

- Store the product in a dry, ventilated place between 15 and 30°C and relative humidity below 60%.
- Do not store in the refrigerator.
- Return the syringe properly closed, along with the silica sachet, inside the packaging, which should be correctly closed.
- Upon subsequent use, check if the silica sachet inside the envelope remains blue.

DISPOSAL METHOD

Dispose of the product according to national laws and recommendations from local authorities for proper disposal. Ready-to-use bioceramic paste for intracanal dressing.

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