MTA-FILLAPEX

CHOOSE YOUR LANGUAGE

ENGLISH	3	HRVATSKI	38
ESPAÑOL	6	SLOVENŠČINA	41
PORTUGUÊS	9	SLOVENSKY	44
FRANÇAIS	12	POLSKI	48
ITALIANO	16	LIETUVIŠKAI	51
ROMÂNĂ	19	ČESKY	54
NEDERLANDS	22	MAGYAR	57
DEUTSCH	26	TÜRKÇE	61
SVENSKA	29	БЪЛГАРСКИ	64
NORSK	32	ΕΛΛΗΝΙΚΑ	67
SUOMI	35		

ENGLISH

Bioceramic root canal sealer

DEFINITION

MTA-FILLAPEX is a bioceramic root canal sealer based on MTA (Mineral Trioxide Aggregate).

INDICATIONS

Sealing the root canal of permanent teeth.

INTENDED USE

It is intended for use in patients who require endodontic procedures, with the function of sealing the root canal.

INTENDED PUBLIC

Patients with closed apex permanent teeth diagnosed with pulp or periapical disease.

INTENDED USERS

MTA-FILLAPEX should only be used by dental surgeons trained to perform endodontic procedures.

INTENDED ENVIRONMENT

Dental practices.

CONTRAINDICATIONS

- MTA-FILLAPEX contains components that may cause reactions in susceptible individuals. Do not use on patients who are allergic to the product's components.
- Do not use the product after its expiration date.
- Do not use the product if its packaging is damaged.

COMPOSITION

- Base Paste: Salicylate Resin, Natural Resin, Calcium Tungstate, silicon dioxide, and titanium dioxide.
- Catalyst Paste: Sulfonamide Resin, Natural Resin, Mineral Trioxide Aggregate, silicon dioxide, and titanium dioxide.

TECHNICAL INFORMATION

- High alkalinity: inhibits bacterial growth;
- Adequate radiopacity;
- Adequate flowability;
- Does not stain the dentinary structure;

- Does not contain eugenol, therefore does not interfere with resinous restoration materials;
- Safe, atraumatic and biocompatible material for root canal filling procedures;
- The working time is 23 minutes.

WARNINGS

- Contact with eyes or skin: you must avoid contact of both the separate and already mixed pastes with eyes or skin. In case of accidental contact, wash abundantly with water;
- Avoid contact with oral mucosa. In case of contact, rinse with water and avoid ingesting the product. If any sensitivity persists, seek medical attention immediately.

Note: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority health in which the user and/or patient is established

PRECAUTIONS

- During the use of this product, it is recommended the patient and professional use suitable protective clothing, eye protection and gloves
- Use the product only after controlling the infection or the material will not function correctly due to the acidic pH of the environment;
- Ensure that the covers of the base and catalyst pastes are not interchanged, because replacing them incorrectly can cause the hardening of the product inside the tube;
- If the syringe is contaminated with saliva or blood during the application, dispose of the syringe and do not use it on other patients;
- The syringe and tube must be cleaned immediately after use with alcohol and instruments must be sterilized;
- The safety and efficacy of this product have not been established in pregnant, lactating women or for children and adolescents.
- Keep out of reach of children.

TECHNIQUE OF USE

Dosage and mixing:

- Double syringe: The double syringe ensures a 1:1 mixing ratio. Press the plunger and dispense the material directly onto a glass plate, mixing block, or directly into the canal. The product must be used immediately after dispensing.
- Tubes: Dispense equal amounts (1:1) of the pastes and spatulate for 30 seconds to homogenize the mixture.

ROOT CANAL FILLING OF PERMANENT TEETH

LATERAL COMPACTION:

1. Anesthetize, install absolute isolation, perform the coronal opening, and then perform biomechanical preparation of the canal;
2. Perform the final irrigation of the canal with sodium hypochlorite and EDTA;
3. Select the appropriate digital spreader;
4. Dry the canal using only paper cones;
5. Mix the cement with equal amounts (1:1) of the base and catalyst pastes and spatulate for 30 seconds to homogenize the mixture. When using the double syringe, dispense the material directly onto a glass plate, mixing block, or directly into the canal;
6. Introduce the gutta-percha cone into the canal according to the last file used;
7. With the main cone in place, insert the spreader to create space for inserting accessory cones loaded with obturation cement;
8. Repeat this process until the spreader no longer penetrates;
9. Take an x-ray to verify the correct filling of the canal;
10. Cut the cone to the desired height with heated instruments followed by vertical compaction;
11. Remove excess material from the canal walls with water, perform coronal sealing, and restoration.

SINGLE CONE TECHNIQUE:

1. Anesthetize, install absolute isolation, perform the coronal opening, and then perform biomechanical preparation of the canal;
2. Perform the final irrigation of the canal with sodium hypochlorite and EDTA;
3. Dry the canals with absorbent paper cones;
4. Mix the cement with equal amounts (1:1) of the base and catalyst pastes and spatulate for 30 seconds to homogenize the mixture. When using the double syringe, dispense the material directly onto a glass plate, mixing block, or directly into the canal;
5. Introduce the gutta-percha cone loaded into the canal;
6. Take an x-ray to verify the correct filling of the canal;
7. Cut the cone to the desired height with heated instruments followed by vertical compaction;
8. Remove excess material from the canal walls with water, using appropriate irrigating solutions for this purpose, and perform coronal sealing and restoration.

STORAGE

- Keep in dry and cool place, away from humidity and sources of heat;
- Do not store in the refrigerator;
- Keep the product tightly capped.

DISPOSAL METHOD

The product must be disposed of in accordance with local regulations.

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